


Installing on Android

PacSana app can be installed via the Google Play store, so we start by tapping on the Play Store app icon.



[This Photo](#) by Unknown Author is licensed under [CC BY-SA](#)


In the search box type, Pacsana, then tap the Enter or search key denoted in the image as a magnifying glass.

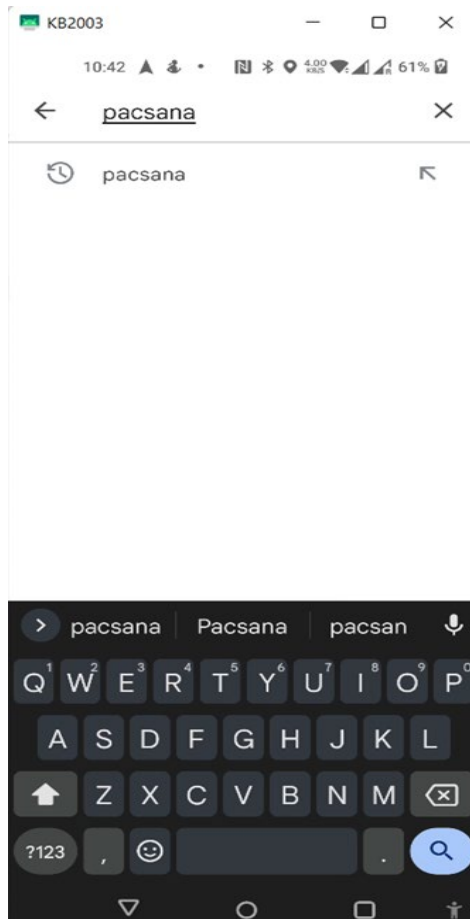


One of the many results returned will be “PacSana, DunavNET – Health & Fitness”, tap on the PacSana icon.




Then tap in install button on the next screen

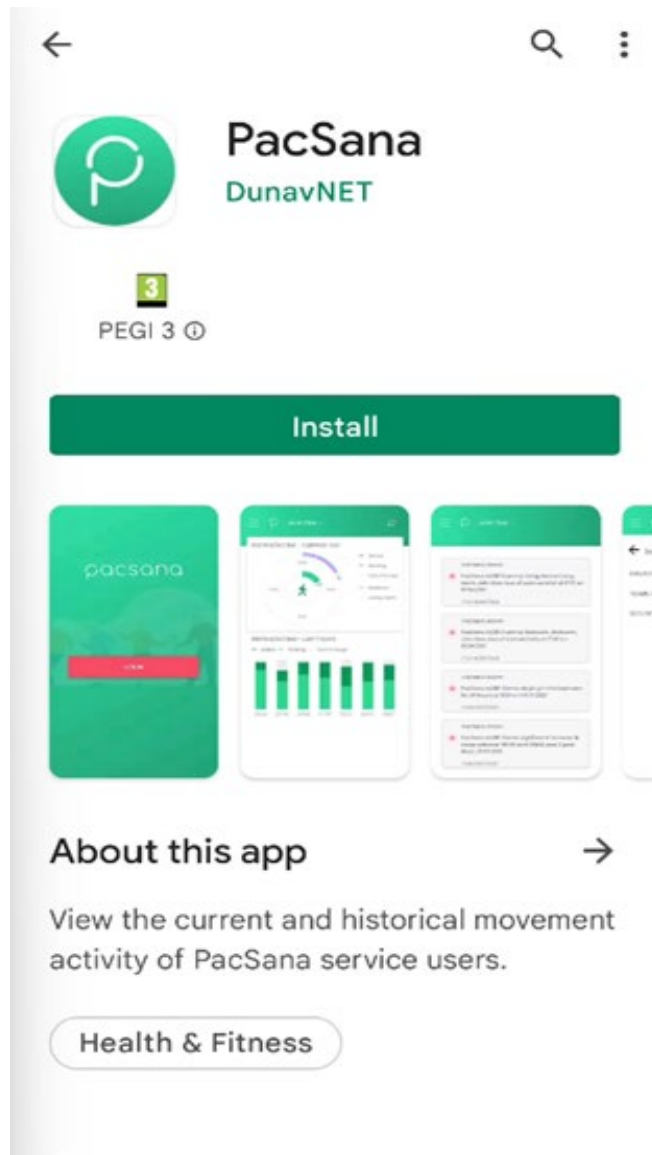

After a short time, the app will be downloaded and installed on your device. Tap on the option to Open.

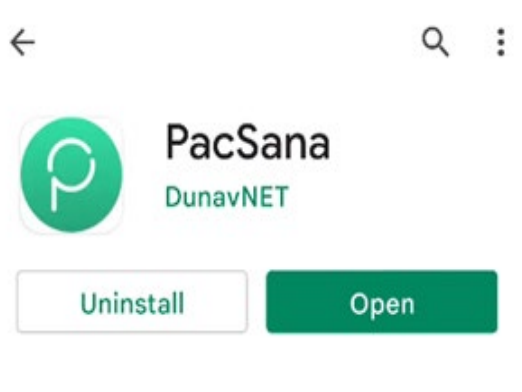

You will now be presented with the PacSana logo and login button.

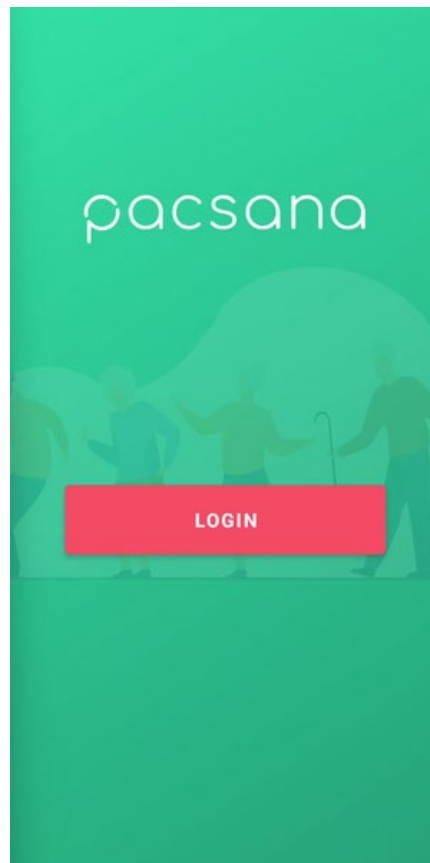

A login screen will appear; type in your username and password provided by a member of the Pacsana support team (support@pacsana.com) then tap the enter key.